

Getinge Online

Anytime. Anywhere.

Getinge Connect module enables secure wireless transfer of service and equipment data from Flow Anesthesia system to a customer web portal called Getinge Online. A wireless connectivity solution enabling you, your team and hospital, to gain clinical insights and maximize uptime. All to improve efficiency in the operating room.

Connected medical devices

By connecting your Getinge devices, you allow your department to make use of intelligence and knowledge within your equipment. A digital service developed in close collaboration with our customers to help you transfer and access data whenever you want. Wherever you are.

How?

Connected Services offers the possibility to connect your Flow Anesthesia equipment by using three different networks: LAN, Wi-Fi and LTE/4G.

Getinge Connect module is a stand-alone connectivity module installed on the Flow Anesthesia system.



LAN

With LAN, the Getinge Flow Anesthesia system connects to the hospital LAN socket to access the hospital LAN. The Flow Anesthesia system can communicate either directly from the device or via the Getinge Connect module.

Wi-Fi

With Wi-Fi, the Flow Anesthesia system connects wirelessly to the hospital LAN. To be able to connect via Wi-Fi, Getinge Connect module must be used.

LTE/4G

With LTE/4G, the Flow Anesthesia system connects wirelessly to the Connected Services server via the LTE/4G network and not the hospital LAN.

To be able to connect via LTE/4G a Getinge Connect module with a factory installed SIM card must be used.


How does it work?

Getinge Connect module enables flexible secure wireless transfer of equipment data from Flow Anesthesia system to the customer web portal Getinge Online. In Getinge Online, hospital personnel can view and analyze service and equipment data regarding their equipment, combined with helpful documentation.

The user interface for Getinge Connect module is integrated in Service & Settings in the Flow Anesthesia system.

For further information please refer to:

- Getinge Connect module IFU
- Getinge Connect module – Service Manual including installation instructions
- Getinge Connected Services – Flow-Family – White paper

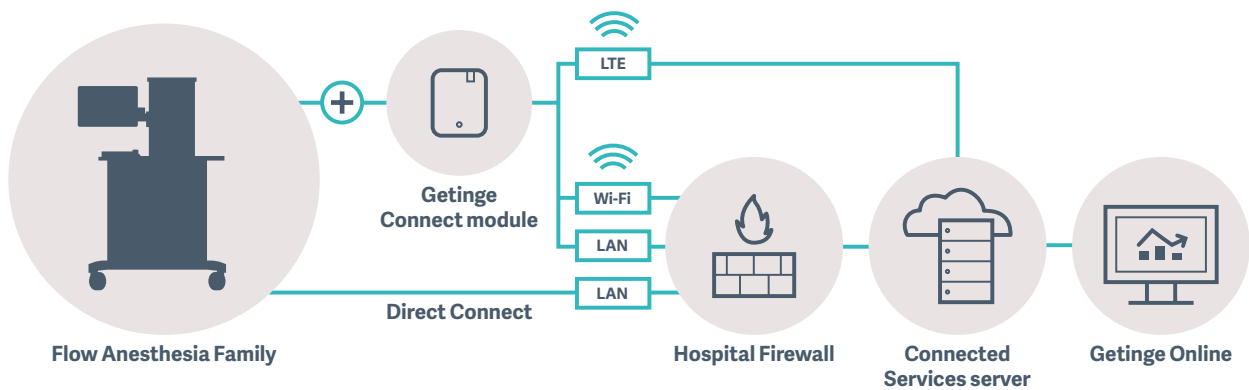

Item code

- Getinge Connect module X10 (enabling Wi-Fi)
Article number: 6889600
- Getinge Connect module X20-EU (enabling Wi-Fi and LTE/4G)
Article number: 6889300



The document is intended to provide information to an international audience outside of the US.

Depending on national regulations, the product offering and indications can differ from market to market, please check with your local Getinge sales team.

Manufacturer · Maquet Critical Care AB · Röntgenvägen 2 SE-171 54 Solna · Sweden · +46 (0)10 335 73 00

www.getinge.com

GETINGE 