

Indication by design

With millions of chest drains sold worldwide, Getinge knows that one chest drain does not necessarily fit all applications. Getinge's innovative Indication By Design platform sets a new bar for clinical performance, providing a full line of chest drainage products designed to meet the needs of the most challenging cardiac, thoracic, and pediatric patients.

Getinge offers a complete assortment of high-performance chest drains tailored to suit your specific needs. Each drain is crafted of quality materials, each upholding the Getinge commitment to patient safety and comfort.

Getinge also offers a variety of accessories to secure any size thoracic catheter to adult, pediatric or mobile chest drains.

Your application. Your choice.

Need Assistance?

For toll free 24-hour technical support, call Getinge at 1 800 528 7486.

Clinician Education

Getinge provides continual education in the form of: handbooks, wall charts, videos, competency manuals, free continuing education credits for nurses, and website with information available for downloading. Visit www.getinge.com/draineducation to explore all Getinge has to offer.



Getinge Thoracic Drainage

Clinician's guide to therapeutic devices

GETINGE ✱

Getinge is a registered trademark of Getinge AB, its subsidiaries, or affiliates in the United States or other countries •
⚠ CAUTION: Federal (US) law restricts this device to sale by or on the order of a physician. Refer to Instructions for Use for current indications, warnings, contraindications, and precautions. • 011064 Rev AB

Sales Office - 45 Barbour Pond Drive - Wayne, NJ 07470 - USA

www.getinge.com/draineducation

MCV00071105 REVA
EXP06/20

GETINGE ✱

Getinge Thoracic Drainage

A clinician's guide for initiating, collecting and transitioning the patient through all stages of therapeutic treatment in the ER, OR, ICU, at bedside, and in transport.

Initiation


Where are you inserting the catheters anatomically?

- Mediastinal? Select the Getinge thoracic mediastinal catheter.
- Pleural? Select straight or right-angle Getinge thoracic catheter.

Depending on the insertion location, select the appropriate Getinge thoracic catheter softness or firmness (either thermo-conforming PVC, firm PVC, or silicone).

Which department in the hospital/facility are catheters being placed in patients?

- Pre-hospital, ER, ICU or bedside? Select the all-in-one Getinge PleuraGuide Chest Tube Insertion Kit and appropriate catheter.
- OR? Use the selected Getinge thoracic catheter in the operating room.

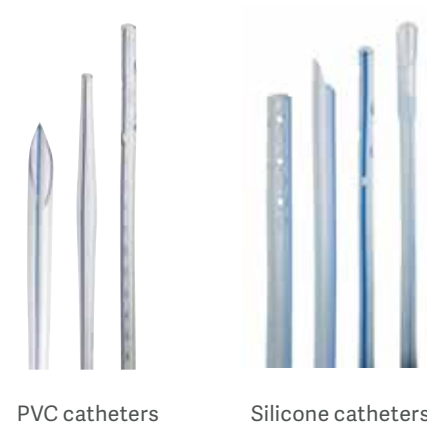

PVC catheters

Silicone catheters

Getinge Thoracic Catheters

Improving patient outcomes starts with high-performance thoracic and mediastinal catheters — designed for patient safety and comfort.

- Thermo-conforming PVC catheters
- Firm PVC catheters
- Silicone catheters

Getinge Chest Tube Insertion Kit

Three-level designed kit allows for quick access to all 27 components, in procedure order, housed in a single compact package.

- PleuraGuide Chest Tube Insertion Kit



PleuraGuide Chest Tube Insertion Kit

Collection

What type of drain do you need?

- Wet suction water seal? Select Atrium Ocean Chest Drain.
- Dry suction water seal? Select Atrium Oasis Chest Drain.
- Dry suction dry seal? Select Atrium Express Chest Drain.
- Dry suction dry seal mobile drain? Select Atrium Express Mini 500 Chest Drain.
- Manual suction with a mobile drain? Select the Express Suction Bulb for use with Express Mini 500.
- Step down or mobile drain? Select the Atrium Pneumostat Chest Drain Valve.

Atrium Chest Drains

The innovative portfolio of Getinge chest drains are designed to meet the needs of the most challenging cardiac, thoracic, and pediatric patients — and their changing needs during therapeutic treatment.

- Ocean Wet Suction Water Seal*
- Oasis Dry Suction Water Seal*
- Express Dry Suction Dry Seal (single and ATS only)
- Express Mini 500 Dry Suction Dry Seal Mobile Drain
- Express Suction Bulb
- Pneumostat Chest Drain Valve

* Available in single collection, dual collection, infant/pediatric, and autotransfusion (ATS) configurations.



Atrium Oasis Chest Drain



Atrium Express Mini Chest Drain



Pneumostat Chest Drain Valve

Transition

How many drains are you connecting?

- Connecting two drains to one catheter? Select appropriate Y-Extension, Y-Adapter, Y-Connector.
- Connectors – secure any size thoracic catheter to any drain.

Are the original tubes compromised?

- Select the Patient Tube Pack for replacements.

Is the patient ready to step-down?

- Select any appropriate connector to attach the catheter to an Atrium Chest Drain.


Are you ready to remove the drain(s)?

- Select the Red Cap for safe disposal of drain(s).

Accessories

Based on the varying needs of thoracic drainage patients, a full line of Getinge thoracic drainage accessories provides flexibility to attach any Getinge Chest Drain to multiple catheter types creating a variety drainage configurations.

- Y-Extension, Y- Adapter, Y-Connector
- Connectors
- Patient Tube Pack
- Red Cap

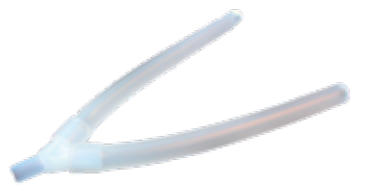

Suction Y-Adapter



Patient Tube Pack



Step Down Connector



Red Cap