
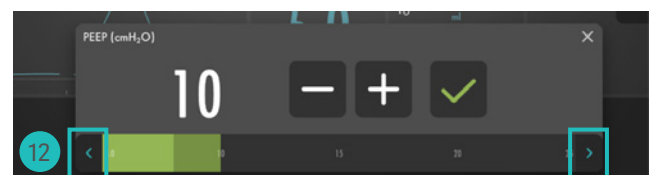


Servo-c

– short guide

Setting up the Servo-c

1. Plug in the power cord.
2. Switch ON the ventilator.
NOTE: When switching on the Servo-c, you need to first pull the ON/OFF switch downwards.
3. Connect the oxygen and air hoses.
4. Lock the wheels on the cart.
5. Start the **PRE-USE CHECK**. Follow on-screen instructions. (You need the test tube during the Pre-Use Check). The patient circuit test is included in the Pre-Use Check, but can also be selected separately.
6. Choose **PATIENT CATEGORY**: adult or pediatric.
7. Choose **VENTILATION TYPE**: invasive or non-invasive (optional).
8. Calculate Predicted Body Weight (PBW) automatically by tapping **PBW** and enter patient gender and height. (For pediatric patient category it is the patient's weight). Tidal Volume in ml/kg will now be monitored.
9. Tap **VENTILATION MODES**. (If you tap and hold on the mode you will get more information).
10. Choose mode and adjust the settings.
11. Some settings have dynamic images.
12. The safety scale can be expanded by tapping **>**.
13. Change the alarm limits in alarm window.
14. Tap **START VENTILATION**.

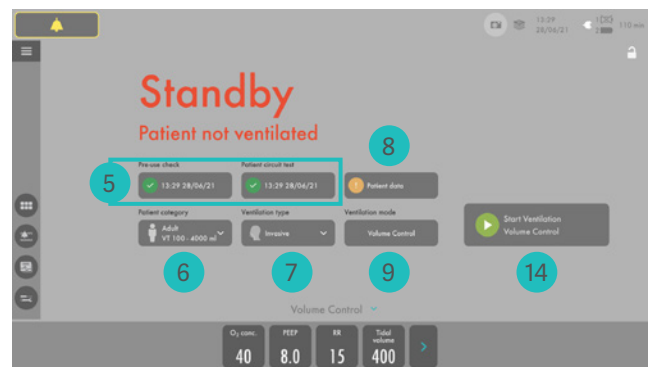

Refer to user's manual for more detailed information

NOTE: Alarms can be in one of three colors depending on priority:

- High priority**
- Medium priority**
- Low priority**



Alarms audio paused
 – Active alarms can be silence for two minutes by tapping.



Pre-silence alarms
 – To silence most alarms for two minutes, tap when no alarms are active.

The arrow shows the actual measured value for the patient.

Autoset Alarm limits in controlled modes



Alarm volume



Tap on the active alarm and you can adjust alarm limits directly.

Menu

Modes

Alarm settings

Trends

Maneuvers:

Manual breath, static measurement, nebulization and configure O₂ boost level.



Additional values and settings

O₂ boost:
Tap and hold

Disconnection

Stop ventilation:
Tap Standby then tap and hold

This document is intended to provide information to an international audience outside of the US. Servo-c may be pending regulatory approvals to be marketed in your country. Contact your Getinge representative for more information.
Manufacturer · Maquet Critical Care AB · Röntgenvägen 2 SE-171 54 Solna · Sweden · +46 (0)10 335 73 00